

Kurzanleitung

Voraussetzung für das Registrieren der Software ist:

- Ein Funktionierender Internet-Anschluss
- Eine gültige E-Mail Adresse



Starten Sie das Programm WinSpiroPRO, indem Sie auf das Icon  doppelklicken.

Folgendes Meldefenster wird ausgegeben:




Drücken Sie die Taste JETZT REGISTRIEREN.

Das folgende Meldefenster wird ausgegeben:



Klicken Sie mit der Maus auf die Taste WEITER.

Das folgende Fenster wird ausgegeben:

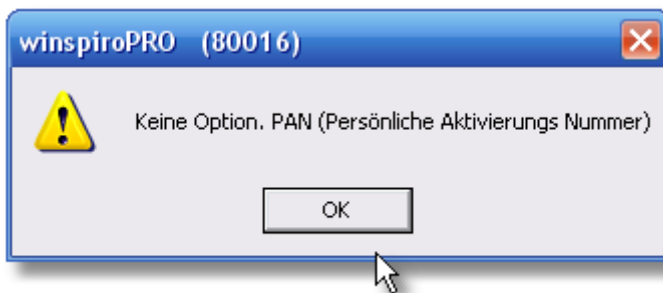

Drücken Sie auf den Knopf OK.

Darauf wird die Internetverbindung zu einem Formular aufgemacht.

The screenshot shows the registration interface for WinSpiroPRO. On the left, there is a section for existing users with a 'Login' button. The main area is for new registration, titled 'NEUE REGISTRIERUNG'. It includes a form with fields for 'Vorname', 'Nachname', 'E-Mail', 'Ihr Passwort', and 'Land'. There are also fields for 'Gerätetyp/Ihr Gerätermodell' and 'Seriennummer'. A 'Zusätzliche Daten' button is located below the main form. A warning icon and text indicate that registration information will be emailed. At the bottom of the form is a 'Bestätigen' button. On the right side, a vertical list of languages is provided for selection, including Čeština, Dansk, Deutsch, English, Español, Français, Italiano, Magyar, Nederlands, Polski, Português, Русский язык, Română, Shqipë, Slovenčina, Türkçe, Ελληνικά, Български език, Српски, עברית, العربية, فارسی, ไทย, 한국어, and 中文. The footer contains the text: © 1996 - 2010 MIR - Medical International Research via del Maggolino, 125 00155 - Roma, Italy.

Machen Sie bitte Ihre Angaben. Besonders wichtig ist Ihre E-Mail Adresse.

Klicken Sie auf den Knopf BESTÄTIGEN.

Sie erhalten in Kürze per Mail Ihre Aktivierungsnummer.

Beenden Sie am besten das Programm WinSpiroPRO und starten es neu.

Kopieren Sie von der E-Mail die Aktivierungsnummer und fügen Sie sie in das PAN-Feld ein.

Direkt zum Anmeldeformular gelangen Sie unter folgender Adresse:

<http://www.spirometry.com/online/Startregistration.asp?page=/Default.asp>

Fertig. Viel Freude mit Ihrer neuen Spirometrie-Ausrüstung wünscht Ihnen

Rolf-R. Deckert